Panasonic

Operating Instructions 使用說明書

Upper Arm Blood Pressure Monitor 上臂式血壓計

Model No. **EW-BU15**

English	1
繁體中文	T 1

Before operating this device, please read these instructions thoroughly and keep this manual for future use.

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Table of contents

BEFORE USE

Important safety instructions.....	2
Parts identification	3
Display	5
Inserting batteries	6
Setting the time and date	7
Wrapping the cuff	8

HOW TO USE

Getting into the measurement position	9
Taking a reading (Measuring/Recording measurements)...	10
About each mark.....	11
Calling up readings from memory.....	13
Deleting all data stored in memory.....	13
Comparing with old measurements.....	14
Comparison with past data	14
Storage	15

APPENDIX

Care and maintenance.....	16
Specifications	16
Explanation of rating plate symbols	16
Q&A	17
Troubleshooting	18

Important safety instructions

- * Before using this instrument, please read the “IMPORTANT SAFETY INSTRUCTIONS” and be sure to use the instrument properly.
- * The cautions in this section are intended to instruct you in the proper use of this instrument and prevent danger or injury to yourself or other people.

All of the cautions contain important information regarding safety and must be followed.

WARNING: Failure to follow the caution may result in death or serious injury of people.

CAUTION : Failure to follow the caution may result in injury to people, or in damage to property.

WARNING :

- Persons with serious obstructions of blood flow in the arm should consult a doctor before using this instrument. Failure to do so may result in deterioration of physical condition.
- Do not let this product be used by children, people who cannot express their feelings, or who cannot operate the product for themselves. Do not use the unit on an arm where a catheter has been inserted. Such use may result in an accident or injury.
- Do not spill water on the main unit. Doing so may cause electric shock, short-circuiting, or product damage.
- Never modify the product, or disassemble or attempt to repair it yourself. Doing so may result in fire or in injury due to abnormal operation.


CAUTION :

- Stop using the device if it does not operate or when you sense a malfunction, immediately remove the batteries and have the device inspected and repaired. If this is not observed, an accident, electric shock or fire might occur.
- Do not make judgements yourself regarding the measurement results: self-treatment is not recommended.
 - Take measurements according to your doctor’s instructions and have your doctor do the diagnosis.
 - Take medicine according to your doctor’s instructions.
- Do not use for purposes other than the measurement of blood pressure. Doing so may result in an accident or malfunction.


Parts identification

Main unit

Left side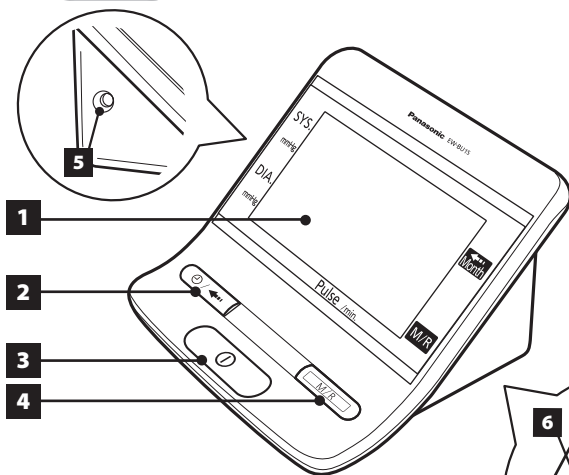

Bottom

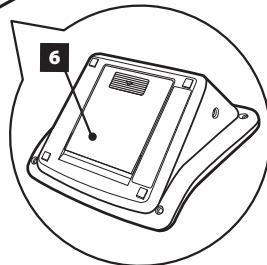1 Display


3 Off/On & start button

5 Cuff socket

2 Clock/Comparison
button

4 Memory/Recall
button

6 Battery cover

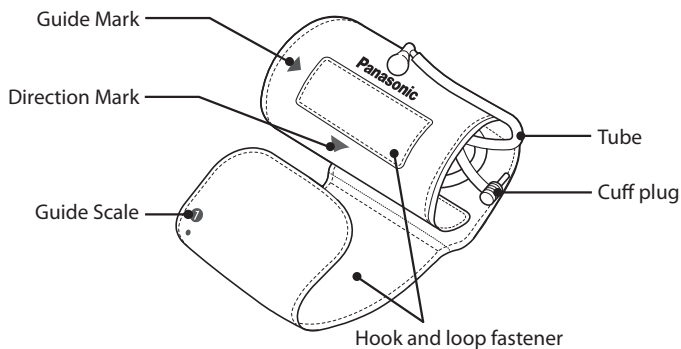
Cuff**Includes**

Main unit
Cuff
Batteries
Pouch
Operating instructions
Guarantee card

Accessories**Pouch**

* Specifications of the product or parts may be changed without prior notice.

Display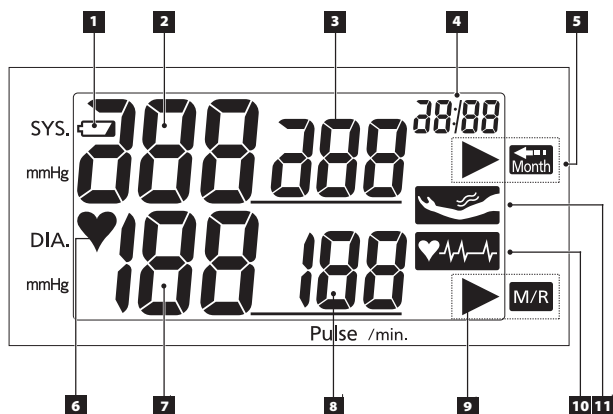

1 Battery icon

Notifies when running out of battery

2 Systolic blood pressure

Displays measured or stored value

3 Systolic blood pressure from previous data

Displays the monthly average of systolic blood pressure from previous data

4 Date/Time/Memory number/Measured time of diastolic blood pressure displayed during comparison of previous data

5 Comparison mark

Displays when comparing previous data

6 Pulse indicator

Displays pulse rhythm

7 Diastolic blood pressure

Displays measured or stored value

8 Pulse rate/Diastolic blood pressure from previous data

- Displays measured or stored pulse rate
- Displays the monthly average of diastolic blood pressure when comparing previous data

9 Record mark

Displays when the measurement can be recorded or when the recorded measurement is displayed

10 Irregular pulse detection


Displays if irregular pulse is detected during measurement

11 Movement warning

Detects irregular arm movements

Inserting batteries


- * **Always use alkaline batteries. (4 AA-size LR6 alkaline batteries)**
- * When replacing batteries, always use 4 fresh alkaline batteries of the same type from the same manufacturer, and replace them simultaneously.
- * If the product will not be used for a long time, remove the batteries.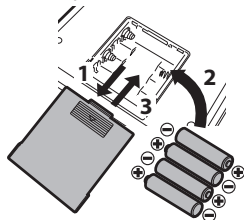

1. **Turn the main unit over and remove the battery cover by sliding it in the direction of the arrow while pressing lightly on .**
2. **Check the polarities of the batteries and install them correctly.**
3. **Replace the battery cover and slide it closed.**

Replace batteries when :

- The “low battery” indicator is displayed.
- No display appears even if  is pressed.

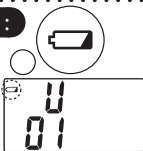CAUTION :

- Always follow the cautions printed on the batteries.
- Check the polarities of the batteries and install them correctly.
- Remove exhausted batteries promptly.
- If the product will not be used for a long time, remove the batteries.
- Batteries should be used before their expiration date. Use after the expiration date may result in injury or staining of the surrounding area due to generation of heat, bursting, or leakage.

For Batteries

- Batteries last for approx. 500 times, which is 3 times/day for 5 months (when using Panasonic alkaline batteries AA cell LR6). (Under the condition that the room temperature is 23°C, 170mmHg pressured, and the length of upper arm is 30cm.)
- If batteries other than alkaline batteries are used or batteries are used in a cold room, or measuring a person with high blood pressure, battery life may be dramatically shorter.
*Use alkaline batteries. (Panasonic Oxyride batteries can also be used.)
- Be sure to replace batteries once a year to prevent battery leakage.

Setting the time and date

- * Please start setting the date and time under the condition that the power is off.
- * Please be sure to set the date and time before taking the first measurement after purchasing the blood pressure monitor or after changing the batteries.
- * During setting, if no button is pressed for 5 minutes or more, the clock will be displayed as "--:--", and the power will be off without saving any settings. It is also possible to perform measurement and recording even under this circumstance. However, the record will not include any time information.

Adjust the flashing numbers sequentially

Adjust numbers

Complete setting

Hold

"year"

"month"
mm/dd

"date"

"hour"

"minutes"

1. Hold  (for 3 seconds or longer). "year (2011)" will flash.

2. Adjust the "year".

- (1) Press  to set the desired number.
- (2) Press  to complete the setting.

- Setting range for years is from 2011 to 2050.

3. Adjust the "month" --> "date" --> "hour" --> "minutes" by repeating procedures in step 2.


4. Press  to complete the settings.

- This setting cannot be completed by pressing  before adjusting the "hour" and "month". Be sure to press  after adjusting the "hour" and "month" to complete settings.
- The set time will disappear after 5 seconds.
 - * To reset, perform the procedures in step 1. to 4.

Wrapping the cuff

The cuff can be used on either upper arm. This section shows an example of wrapping the cuff on the left arm.

* The difference in blood pressure between left and right arm may be around 10 mmHg. Be sure to always use the same arm. It is recommended to use the arm with the higher blood pressure.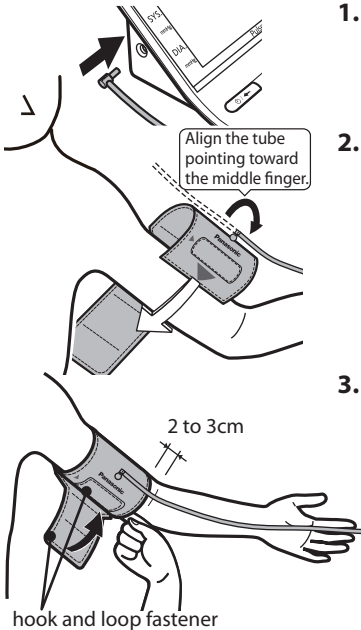

1. Insert the plug into the socket all the way in.

2. Fit the cuff to the upper arm.


- Fit the preformed cuff onto your upper arm with the direction mark at the inner side.
- Wrap the cuff on bare skin.

3. Wrap the cuff.


- Align the tube pointing toward the middle finger.
- Align the rotating part of the tube to the line extended from the middle finger.
- Set apart 2 to 3 cm from the elbow joint.
- Wrap the cuff with a gap of 1 or 2 fingers. Be careful that the cuff is not too tight.
- Press the hook and loop fastener down well to fasten the cuff.

Points when wrapping

Be sure to always wrap on the same position by using the guide.
(if your weight / arm circumference did not change significantly.)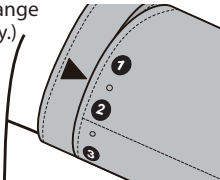

The cuff can be used on either upper arm.
<Example for wrapping on the right arm.>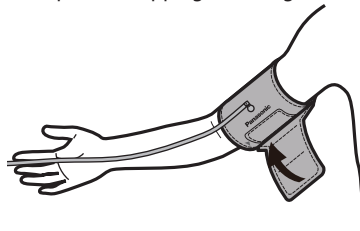


Getting into the measurement position

Sit quietly and relax while taking measurements.

Take a measurement while sitting at a desk and positioning the cuff at heart level as shown in the image below.

* Differences in height will cause differences in measured blood pressure values.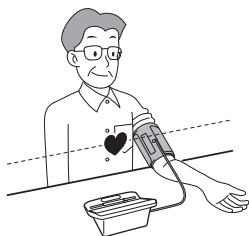

1. **Place the elbow on the table or on something similar.**
2. **Align the height of the cuff to the heart.**
3. **Relax with your palm facing up.**

Use the same upper arm with the same posture at the same time every day. We recommend three measurements a day.

- First... Upon Waking (Because, with almost no influence from body movements, the blood pressure is closest to that at night.)
- Second... Before Lunch (Because, with no influence from the meal, it provides blood pressure values close to the highest of the day.)
- Third... At Bedtime (So as to obtain data close to your average blood pressure values during the day.)

This way, by taking measurements morning, noon and night, you can use the unit to learn the patterns of and monitor your blood pressure.

In the following situations, measurements might be abnormally influenced.

- Within 1 hour after eating or drinking alcohol
- Immediately after drinking coffee or tea, or smoking
- In a moving vehicle
- Immediately after exercising or taking a bath (Wait at least 20 minutes and take a measurement while staying quiet.)
- In cold places (Take measurements at room temperatures around 20 °C.)
- When you need to urinate (Wait several minutes after urinating before taking measurements.)

Be sure to comply with the following points for accurate measurements. Not following these points may make it impossible to obtain meaningful measurements:

- Do not move your body, arm, or fingertips.
- Do not touch the main unit or the cuff during measurement.
- Do not talk.
- Do not use devices emitting strong signals, such as cellular phones, near the main unit. The measurement may not be performed correctly due to the signal and this may lead to malfunctions.

Taking a reading (Measuring/Recording measurements)

Please start taking a reading under the condition that the power is off.

Start

①


1. Press .

After the entire display has been lit, the cuff will be pressurized and measurement will be taken automatically.

- The pressure value and the time will be displayed.
- The ♥ mark will flash when pulse is detected during pressurization.

Finish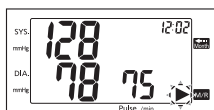


After the measurement is completed, the blood pressure value and pulse rate will be displayed.

- "▶"  is blinking.
- According to the measurement result, safety/warning marks may be displayed. For details, refer to page 11, 12

Memory

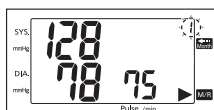M/R

2. To store the measured values,

press .

- "▶"  will go from flashing to being steadily lit.
- The memory number, measurement date, and time are alternately displayed.
- If you do not press , the unit will shut down automatically after approx. 5 minutes.

3. To complete the measurement, press

①

- This unit can store a maximum of 90 measurement values.
- When exceeding 90 stored measurements, stored data will be overwritten, beginning with the oldest data.

About each mark


To perform a correct measurement, this unit provides a movement detection function and an irregular pulse warning function. When the marks appear, the correct measurement might not have been obtained. If any mark appears, it is recommended to repeat the measurement. Stay calm and relax for four to five minutes, before performing a new measurement.

* The "☞" and "⚡" mark will be recorded together with the measurement value.

Movement detection function

The movement detection function is to indicate, with ☞, extra pressure on the cuff caused by movements of the arm during the measurement. When movement is detected and the measurement is completed, the mark is displayed after flashing for approx. 6 seconds.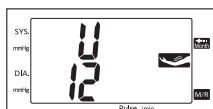

If the movements exceed acceptable levels, no reliable measurement can be performed and the error message "U 12" will be displayed.

Irregular pulse warning function

If the pulse rate varies by more than $\pm 25\%$ from the average while a measurement is being taken, ⚡ will be displayed when the measurement is completed. The mark is displayed after flashing for approx. 6 seconds.

If the ⚡ mark is displayed an irregular pulse rate is detected, which is not necessarily caused by arrhythmia.

- Don't try to interpret readings or attempt to treat any condition yourself. Always follow the guidance of a health professional.
- However if this mark continues to display, consult your physician.

Advice:

In general, blood pressure measurements are lower for the second measurement because you are less nervous and are more tranquil. For people who easily get tense, measuring twice each time is very effective. It is recommended to store all values when performing two measurements or more. When taking two measurements, always stay quiet for 4 to 5 minutes after the first measurement before taking the second measurement.

Reading the blood pressure display


Whether the systolic and diastolic blood pressure values are in the high blood pressure region or normal blood pressure region are judged based on the WHO/ISH (World Health Organization/International Society of Hypertension) blood pressure classifications, and the results are indicated by flashing the measurement value.

Blood pressure readings will flash for approx. 5 seconds when in the high blood pressure range.


- Systolic blood pressure: 140 mmHg and over
 - Diastolic blood pressure: 90 mmHg and over
- (Only when measurement is complete)

WHO/ISH blood pressure classifications*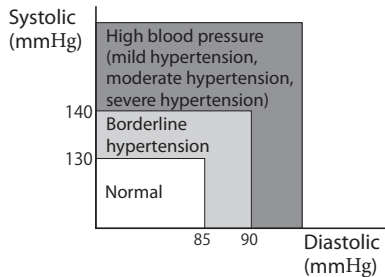

* 1999 World Health Organization-International Society of Hypertension Guidelines for the Management of Hypertension

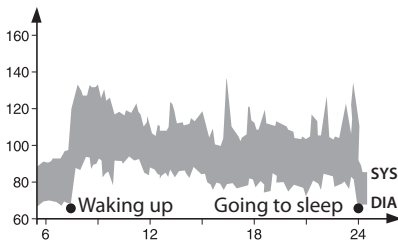Blood pressure is not a fixed parameter!

The blood pressure fluctuations of this example of a day profile show why a single value is not suitable for diagnosis and therapy.

For people with high blood pressure a fluctuation of up to 50mmHg within the course of a day can occur.

Calling up readings from memory

When the unit is turned off, saved readings can be recalled.

M/R

"Latest"

1. Press **M/R**.

The latest record will be displayed.

M/R

"Previous"


2. To display previous data, press

M/R again.

- New data will be displayed every time when pressing **M/R**.

M/R

"Second to last"

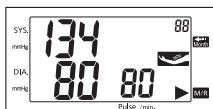⋮

ⓘ

3. To complete the measurement, press

ⓘ.

The power will be switched off automatically after approx. 30 seconds even though without pressing **ⓘ**.

Deleting all data stored in memory

All of the stored measurements can be deleted. The data cannot be deleted individually.

1. Recall stored data using **M/R**.
2. Press **M/R** again and hold it (for approx. 3 seconds) until all readings are deleted.

If everything has been deleted, "000" will be displayed.

Comparing with old measurements

The latest measurement can be compared with the monthly average of the old measurements. The data can be recalled after recording or when the power is off.

- * Correct comparison cannot be done if the date/time is set incorrectly.
- * Comparison cannot be displayed without data.

Displays the past data


Confirm the previous monthly average data

Comparison with past data

The comparison with past data is to display the latest data and the monthly average systolic blood pressure from 1 to 4 months ago, 6 months ago, 9 months ago, and 12 months ago simultaneously. However, the comparison will not include any pulse rate information.


- Press .

 - "Past data comparison" will be displayed.
 - The number on the left indicates the latest record.
 - The number on the right indicates previous monthly average.
 - The number on the top-right corner indicates how many months ago the monthly average is.
- Press .

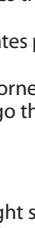
 - The monthly average on the right switches.
 - The monthly average on the right switches from 1 -> 2 -> 3 -> 4 -> 6 -> 9 -> 12 -> 1.
 - Comparison cannot be displayed without data.
- Click  to finish comparison.

 - The power will be switched off automatically after approx. 30 seconds even though without pressing .


"1 month ago"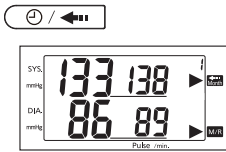


"2 months ago"

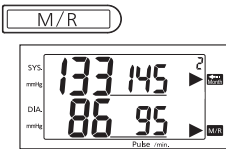
"12 months ago"

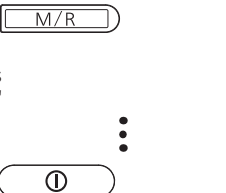
Storage

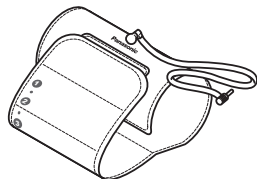1. **Close the hook and loop fastener of the cuff.**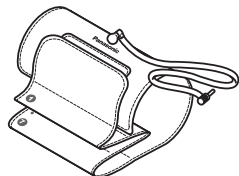

2. **Fold the spare cuff material inside the cuff.**

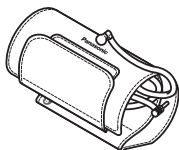3. **Place the tube into the cuff.**
 - Do not wrap the tube around the main unit. It may get damaged by excessive force on the tube.
 - Beware of bending the tube.

4. **Place the main unit and the cuff into the accessory pouch.**
 - To prevent from unexpectedly operations during delivery, do not put the cuff or the accessories on the main unit.

Care and maintenance

- Do not attempt to disassemble, repair or modify the unit.
Doing so may cause fire or cause the unit to malfunction. It may also lead to injury.
- Do not apply excessive force to the unit or drop the unit.
Doing so may cause damage.
- Do not insert dust or foreign objects into the unit.
Doing so may result in damage.
- If the unit is stored at temperatures below the freezing point, do not use it immediately.
Leave it in a warm place for at least an hour before use.
If the unit is not allowed to warm up, it may not pressurize.
- Do not touch the unit when measurement is in progress.
Doing so may cause incorrect readings.
- If the unit becomes dirty, clean it with a soft cloth moistened with warm water or soapy water. (Do not use alcohol, benzine, or paint thinner.)
Use of such chemicals may result in cracking or discoloration.
- When the unit will not be used for a long time (30 days or more), be sure to remove the batteries.
Otherwise, the batteries may leak and damage the unit.
- The pressure cuff cannot be washed.
- For storage, avoid high temperatures, high humidity, and direct sunlight.
Storage under such conditions may result in damage.

Specifications

Power source	DC 6 V (4 AA/LR6 size alkaline batteries)
Display	Digital LCD
Method of measurement	Oscillometric system
Measurement range	Pressure: 0 mmHg to 280 mmHg Pulse rate: 30 to 160 beats per minute
Accuracy	Pressure: Within ± 3 mmHg Pulse rate: Within ± 5 %
Operation temperature/humidity range	10 °C to 40 °C, 30 % to 85 % RH
Storage temperature/humidity range	-20 °C to 60 °C, 10 % to 95 % RH
Measurable arm circumference	Approx. 20 cm to 34 cm
Weight	Approx. 290 g (not including batteries)
Dimensions	7.9 cm \times 13.0 cm \times 11.5 cm
Protection against electric shock	Internally powered equipment, Type BF applied part 

Explanation of rating plate symbols

	Read the operating instructions carefully before use.
	Protection against electric shock Type BF applied part (Blood pressure monitor)
	DC

Q&A

Q Why are the blood pressure values measured at home different from those measured at the doctor's office?

A Blood pressure varies 24 hours a day. In addition, it can change greatly depending on the weather, your mental state, exercise, etc. . In particular, there is the phenomenon of "White coat hypertension" at doctor's offices, where the blood pressure measured in a doctor's office is higher because of stress or worry. **In addition, if the blood pressure measured at home is sometimes higher and sometimes lower than the blood pressure measured at the doctor's office, the following can be considered. Please pay attention to these points when taking measurements.**

1. Is the cuff wrapped firmly?
 - Make sure that the cuff is not wrapped too loosely or too tightly.
 - Is the cuff wrapped around the upper arm? Make sure it is not partly covering the elbow. See page 8 for how to wrap the cuff correctly.
2. Were you worried or upset during measurements?
 - Before taking measurements, take 2 or 3 deep breaths and relax, so that your blood pressure stabilizes.

Q When should I measure my blood pressure?

A We recommend three measurements a day.

First	Upon Waking (Because, with almost no influence from body movement, the blood pressure is closest to that at night.)
Second ...	Before Lunch (Because, with no influence from the meal, it provides blood pressure values close to the highest of the day.)
Third	At Bedtime (So as to obtain data close to your average blood pressure values during the day.)

This way, by taking measurements morning, noon and night, you can use the unit to learn the patterns of and monitor your blood pressure.

- Use of the "Comparing with Old Measurement" function can aid in the detection of a blood pressure pattern known as "early morning surge" in which blood pressure rises rapidly in the morning. (See page14.)

Q Since the  mark was displayed, I retried the measurement by setting my arm in the proper position. But the indicated value is still higher than usual.

A The  mark is just a guide for the proper measurement method.

If you correct your posture when the  mark is displayed but the measured value is still higher or lower than usual, retry the measurement.



The following causes are suspected.

- Did you measure properly (in proper posture)?
Measure as indicated on pages 8–9.
- Measured immediately after smoking.
- Measured when irritated or emotional.
- Measured in cold environments.
- For persons who have a physical constitution with less fluctuation in the pulse rate.
- If you are concerned about the results, retry the measurement after staying quiet for 4 to 5 minutes.

Q The  mark does not light when an arm is moved.

- A** The  mark is a function to notify you that your arm is bent or to indicate extra loads on the cuff.
Therefore, it may not indicate motions (bending an arm slightly, etc.) that do not affect the cuff.
If you are concerned about the result, retry the measurement.

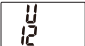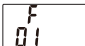Troubleshooting

Display	Status before error	Cause and solution
 is displayed.	The indication disappears in the middle of the operation. (the  mark is displayed.)	▶ Batteries are exhausted. (See page 6.)
 is displayed.	Your arm or body moved.	▶ Has the unit detected movement? (See page 11.)
	Pressurization to more than 280 mmHg was performed. (Cuff pressurizes several times.)	▶ Were measurements taken according to the proper procedure and in the correct posture? (See page 9.)
	Pressure decreased suddenly.	▶
	Cuff does not inflate. The measurement is not possible though the cuff inflates. Cuff pressurizes several times.	▶ <ul style="list-style-type: none"> • Is the cuff wrapped properly? • Is the cuff plug inserted correctly into the socket? (See page 8.)
 is displayed.	Failure of the main unit	▶ Take it to the store where it was purchased for testing and repairs.

Troubleshooting (continued)

Symptom	Possible cause
Systolic or diastolic blood pressure value is high or low.	<ul style="list-style-type: none"> • The position of the cuff is higher or lower than your heart when measuring. • Cuff is not wrapped correctly. • You moved or talked during measurements. (See page9.)
Value is different from that measured by the doctor.	<ul style="list-style-type: none"> • Blood pressure varies in response to minute changes in your mental state, such as your reaction to a doctor's presence. • Sit quietly for 4 to 5 minutes and then take another measurement. (See page17.)
Measurement values are different with each measurement.	
No display appears even when the Off/ On & Start button is pressed.	<ul style="list-style-type: none"> • Batteries are installed with the polarities wrong. (See page6.) • Batteries are exhausted. (See page6.)
The  mark flashes when readings are being taken.	There was a significant fluctuation in pulse when readings were being taken. (See pages 11–12.)
The  mark flashes when readings are being taken.	Your arm or body moved or the buttons are pressed during the measurement. (See page11.)
The  flashes even though I did not move my arm or body.	In some cases this will be displayed when you have tensed the muscles in your hand. Relax the muscles in your hand and take the reading again. (See page11.)
Instrument is broken.	Take it to the store where it was purchased for testing and repairs.

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© Panasonic Electric Works Co.,Ltd.2011
Panasonic Electric Works (Beijing) Co., Ltd.